

Instructions for Use – Sterile Tubing

English

Issue Date:
09/23/2024
Revision: 1

REF B86203 B86200

1. Device Description

Black & Black Surgical supplies a range of sterile tubing to carry out plastic surgery procedures. These devices differ between each other for length and dimension as well as for number and component type.



Carefully read these instructions before using B&B sterile tubing. Keep them in a safe place for future reference



Improper use and handling of the sterile tubes may cause serious consequences

2. Indication for Use

The suction and irrigation tubes are intended for instillation or aspiration of fluid to/from a targeted area of the body during liposuction procedures.

2.1. Intended Users

Black & Black Surgical instruments should be handled and operated by healthcare professionals completely familiar with their use and connection with other medical devices involved in liposuction procedures such as cannulas, canisters and pumps.



Use instruments for their intended surgical purposes only



Federal (USA) law restricts this device to sale by or on the order of a physician

3. Contraindications

Do not use the devices for any intended use other than the one claimed by the manufacturer.

4. Warnings and Precautions



Tubing is latex free

5. Mode of Use



The Black & Black tubes are delivered in sterile condition. Devices are sterilized by means of Ethylene Oxide



Do not reuse the device



Devices cannot be re-sterilized

B86203: Triple Bonded, 3 Port, Aspiration Tubing

- Using sterile technique, remove tubing from pouch
- Within the sterile field, attach the 3 tubes to the rear ports on handpiece.
- Pass other end of tubing outside sterile field

B86200: High Flow Aspiration Tubing

- Using sterile technique, remove tubing from pouch
- Connect one end of tubing to approved canister
- Connect other end of tubing to approved cannula



Do not use the device if packaging is damaged or there is any suspect concerning the sterility of the product

6. Storage

The storage room must be dust-free, low microbiological contamination, dark and free of temperature fluctuations.

7. Used Symbols for IFU and Label

	Symbol for "Manufacturer"	Black & Black Surgical, Inc. 5175 South Royal Atlanta Dr. Tucker Georgia 30084 USA Tel.: 770-414-4880 Fax: 770-414-4879 Email: info@blackandblacksurgical.com Web: http://www.blackandblacksurgical.com
REF	Symbol for "Catalog Number"	
LOT	Symbol for "Batch Code"	
	Symbol for "Consult the Instruction for Use"	
	Symbol for "Do not Reuse"	
	Symbol for "Do not Resterilize"	
	Symbol for "Do not use if packaging is damaged"	
STERILE EO	Symbol for "Sterilized by Ethylene Oxide"	
	Symbol for "Latex Free"	
	Symbol for "Usable Until"	
	Symbol for "Caution, consult accompanying documents"	