

Instructions for Use – Sterile Tubing

English **Issue Date: 6/13/2025**
Revision: 1



B89215
B89217

1. Device Description

Black & Black Surgical supplies a range of sterile tubing to carry out plastic surgery procedures. These devices differ between each other for length and dimension as well as for number and component type.



*Carefully read these instructions before using B&B sterile tubing. Keep them in a safe place for future reference
Improper use and handling of the sterile tubes may cause serious consequences*

2. Indication for Use

The suction and irrigation tubes are intended for instillation or aspiration of fluid to/from a targeted area of the body during liposuction procedures.

2.1. Intended Users

Black & Black Surgical instruments should be handled and operated by healthcare professionals completely familiar with their use and connection with other medical devices involved in liposuction procedures such as cannulas, canisters and pumps.



Use instruments for their intended surgical purposes only



Federal (USA) law restricts this device to sale by or on the order of a physician

3. Contraindications

Do not use the devices for any intended use other than the one claimed by manufacturer.

4. Warnings and Precautions



Tubing is latex free

5. Mode of Use



*The Black & Black tubes are delivered in sterile condition.
Devices are sterilized by means of Ethylene Oxide*



Do not reuse the device



Devices cannot be re-sterilized

B89215: Tubing with Silicone Insert - Double Spike

- Using sterile technique, remove tubing from pouch
- Within the sterile field, attach the male luer tubing connector to the female luer on the hand piece, or directly to a luer lock needle
- Pass other end of tubing with the bag spikes outside sterile field
- Thread the silicone section of the tubing through the head of the infiltration pump
- Close the tubing clamps; attach the bag spikes to the fluid sources
- When ready to use, open the tubing clamp and operate the infiltration pump as per instructions

B89217: Transfer Tubing with Silicone Insert

- Using sterile technique, remove tubing from pouch
- Within the sterile field, attach the open end of the tubing to connector of choice
- Connect spike end of tubing to fluid source.
- Thread the silicone section of the tubing through the head of the infiltration pump
- When ready to use, open the tubing clamp and operate the infiltration pump as per instructions



Do not use the device if packaging is damaged or there is any suspect concerning the sterility of the product

6. Storage

The storage room has to be dust-free, low microbiological contamination, dark and free of temperature fluctuations.

7. Used Symbols for IFU and Label

	Symbol for "Manufacturer"	Black & Black Surgical, Inc. 5175 South Royal Atlanta Dr. Tucker Georgia 30084 USA Tel.: 770-414-4880 Fax: 770-414-4879 Email: info@blackandblacksurgical.com Web: http://www.blackandblacksurgical.com
	Symbol for "Catalog Number"	
	Symbol for "Batch Code"	
	Symbol for "Consult the Instruction for Use"	
	Symbol for "Do not Reuse"	
	Symbol for "Do not Resterilize"	
	Symbol for "Do not use if packaging is damaged"	
	Symbol for "Sterilized by Ethylene Oxide"	
	Symbol for "Latex Free"	
	Symbol for "Usable Until"	
	Symbol for "Caution, consult accompanying documents"	